


MY SURECLICK[®] RESOURCE

A GUIDE TO SUPPORT YOUR
REPATHA[®] JOURNEY

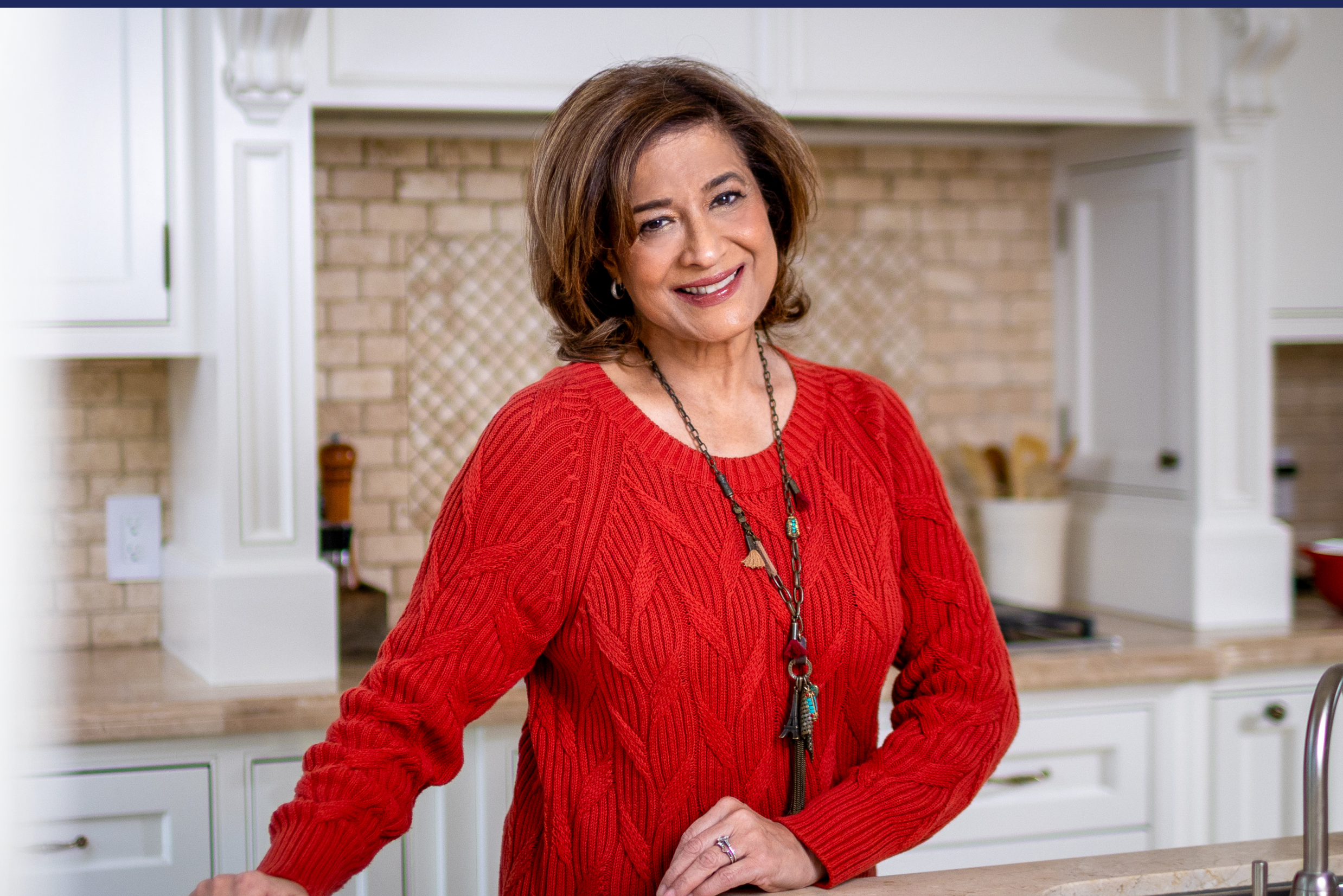

YOUR PARTNER IN CARE, EVERY STEP OF THE WAY

This resource is intended to provide you with an overview and basic information about the SureClick autoinjector and does not replace the information in the **Patient Medication Information** leaflet. Please review the instructions for use included with your Repatha[®] prescription prior to performing your injection.

It is important that you do not try to give yourself the injection unless you have received training from your healthcare provider. If you have further questions, please contact your healthcare provider.

 **Repatha[®]**
evolocumab injection


RepathaREADY[®]
PATIENT SUPPORT PROGRAM

GETTING STARTED WITH YOUR SURECLICK AUTOINJECTOR

This resource is intended to provide you with an overview and basic information about the SureClick autoinjector and does not replace the information in the **Patient Medication Information** leaflet that comes with your Repatha® prescription.

Please review the instructions for use included with your Repatha® prescription prior to performing your injection. If you have further questions, please contact your healthcare provider.

This resource may help you with questions about using the SureClick autoinjector.

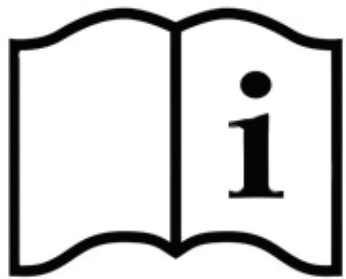

See page 18 to learn how to access the SureClick injection demonstration video!

NEED HELP?  You can call the RepathaREADY® Patient Support Program for assistance at **1-888-Repatha (1-888-737-2842)**, Monday to Friday, 8:00 am to 8:00 pm EST.

 **Visit Repatha.ca**

GETTING
STARTED

INJECT

RESOURCES

FAQ

TRAVEL
GUIDANCE

IMPORTANT
SAFETY INFORMATION

GETTING STARTED AT HOME WITH YOUR SURECLICK AUTOINJECTOR



When you prepare to use the SureClick autoinjector for the very first time, you may wonder if you're going to do it right.

You may have questions or feel uncertain about the process. Other patients feel the same. You are not alone.

There's a lot to learn, and you want to get it right. We're here to support you!

Your Repatha® injection can become a familiar part of your regular routine – one injection, every two weeks.

With this resource and the online tools, **you can do this.**
Now, let's get started!

Carefully review **My SureClick® Resource**, which offers additional helpful information just for you and your caregiver, including:

1. Key injection steps
2. Helpful tips
3. Resources and tools
4. Answers to frequently asked questions

Please see important safety information on page 23. Refer also to the full Patient Medication Information leaflet, which is provided with your medication.

GETTING
STARTED

INJECT

RESOURCES

FAQ

TRAVEL
GUIDANCE


IMPORTANT
SAFETY INFORMATION

FIND A QUIET, CALM PLACE TO INJECT

Start off right and don't rush

Find a quiet and calm place in your home, where pets or other people can't distract or interrupt you.

Consider a room where you can close the door. Sit in a comfortable chair so you feel more relaxed.

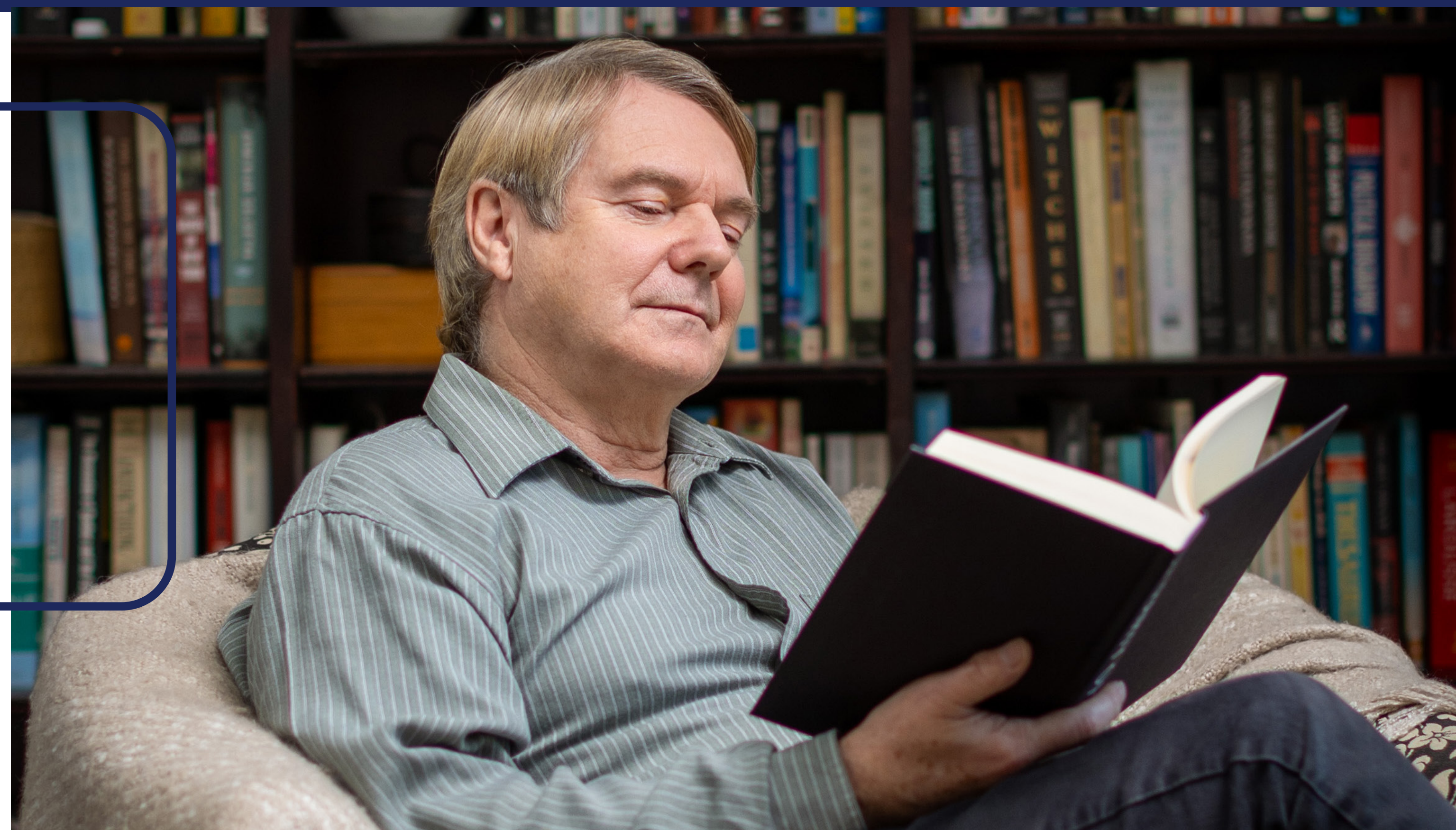

NEED HELP?  You can call the RepathaREADY® Patient Support Program for assistance at **1-888-Repatha (1-888-737-2842)**, Monday to Friday, 8:00 am to 8:00 pm EST.

 **Visit Repatha.ca**

INJECT

In this section, we will discuss the steps for injecting the medicine.

Please read the step-by-step instructions in the **Patient Medication Information** leaflet prior to starting the injection.

Please see important safety information on page 23. Refer also to the full Patient Medication Information leaflet, which is provided with your medication.



GETTING
STARTED

INJECT

RESOURCES

FAQ

TRAVEL
GUIDANCE


IMPORTANT
SAFETY INFORMATION

GET TO KNOW YOUR SURECLICK AUTOINJECTOR

An introduction: Important things to know about your SureClick autoinjector

- SureClick autoinjector is a ready-to-use autoinjector prefilled with your medicine.
- Please read the step-by-step instructions in the **Patient Medication Information** leaflet for detailed information prior to performing your injection.

Guide to parts

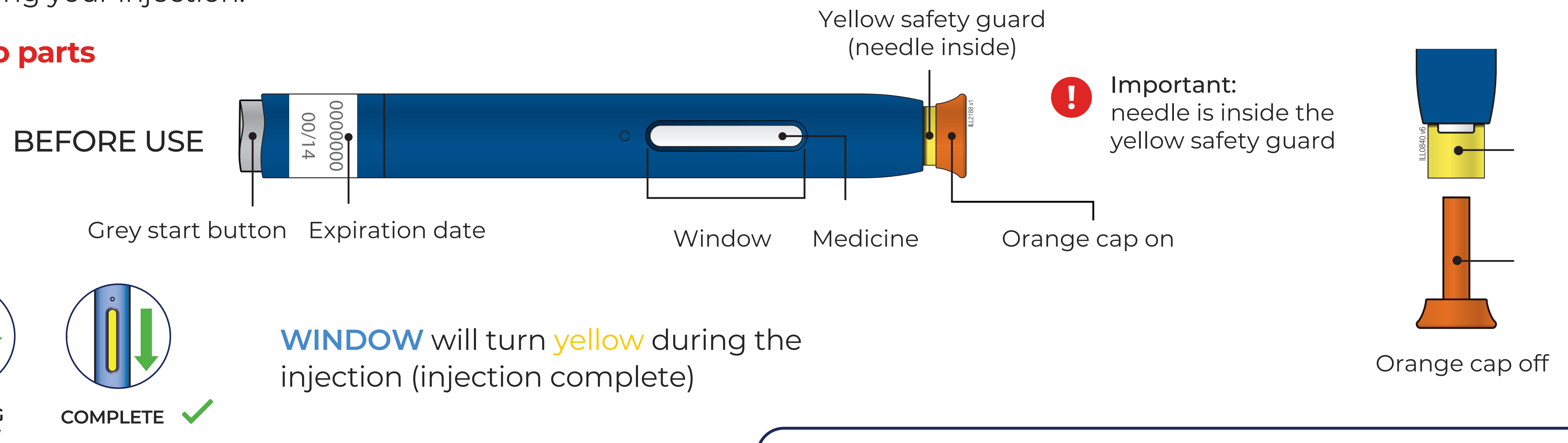

Make sure the **MEDICINE** in the window is clear and colourless to slightly yellow

DO NOT use the autoinjector if the medicine is cloudy or discoloured or contains large lumps, flakes or particles.

DO NOT touch yellow safety guard (needle inside).

DO NOT remove orange cap until ready to inject. Inject within **5 minutes** of removing orange cap.

- If the orange cap is off for more than 5 minutes, this can dry out the medicine and may clog the needle.

NEED HELP?  You can call the RepathaREADY® Patient Support Program for assistance at **1-888-Repatha (1-888-737-2842)**, Monday to Friday, 8:00 am to 8:00 pm EST.

 **Visit Repatha.ca**

PREPARE | READY THE AUTOINJECTOR

Prepare the autoinjector by bringing it to room temperature naturally



Take out one autoinjector that you need for the current injection

- Gather supplies: alcohol wipes, cotton ball or gauze pad, adhesive bandage and a sharps disposal container

⚠ Keep the autoinjector and all medicines out of the reach of children.



Allow the autoinjector to come to room temperature naturally for at least 30 minutes



Use the autoinjector within 5 minutes after removing the orange cap

⚠ **DO NOT** remove the cap until you are ready to inject.

- Waiting longer than 5 minutes can dry out the medicine and may clog the needle when you take the injection.

⚠ **DO NOT** shake the autoinjector.

⚠ **DO NOT** put back in refrigerator once at room temperature.




⚠ Discard the autoinjector if left out of the refrigerator after 30 days.

⚠ **DO NOT** leave the autoinjector in the car.

⚠ **DO NOT** leave the autoinjector in direct sunlight.

⚠ **DO NOT** warm or heat the autoinjector by using a heat source such as hot water or microwave.

⚠ **DO NOT** freeze or use the autoinjector if it has been frozen.

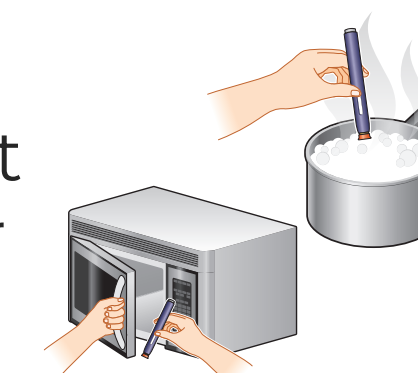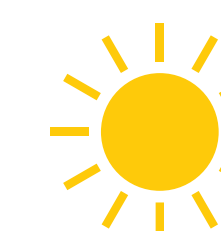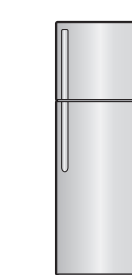


Please see important safety information on page 23. Refer also to the full Patient Medication Information leaflet, which is provided with your medication.

PREPARE | SELECT AND CLEAN THE INJECTION SITE

Select the injection site for you or your caregiver

- Begin by gathering materials in a quiet, calm, and well-lit area. Choose a location to ensure you aren't interrupted. Place supplies on a flat surface.
- Select an injection site from one of the following:
 - Injection site can be your thigh, stomach (except for a 2-inch [5-centimetre] area around your belly button) or the outer area of the upper arm (only if someone else is giving you the injection).
- Wash hands before selecting and cleaning the injection site.

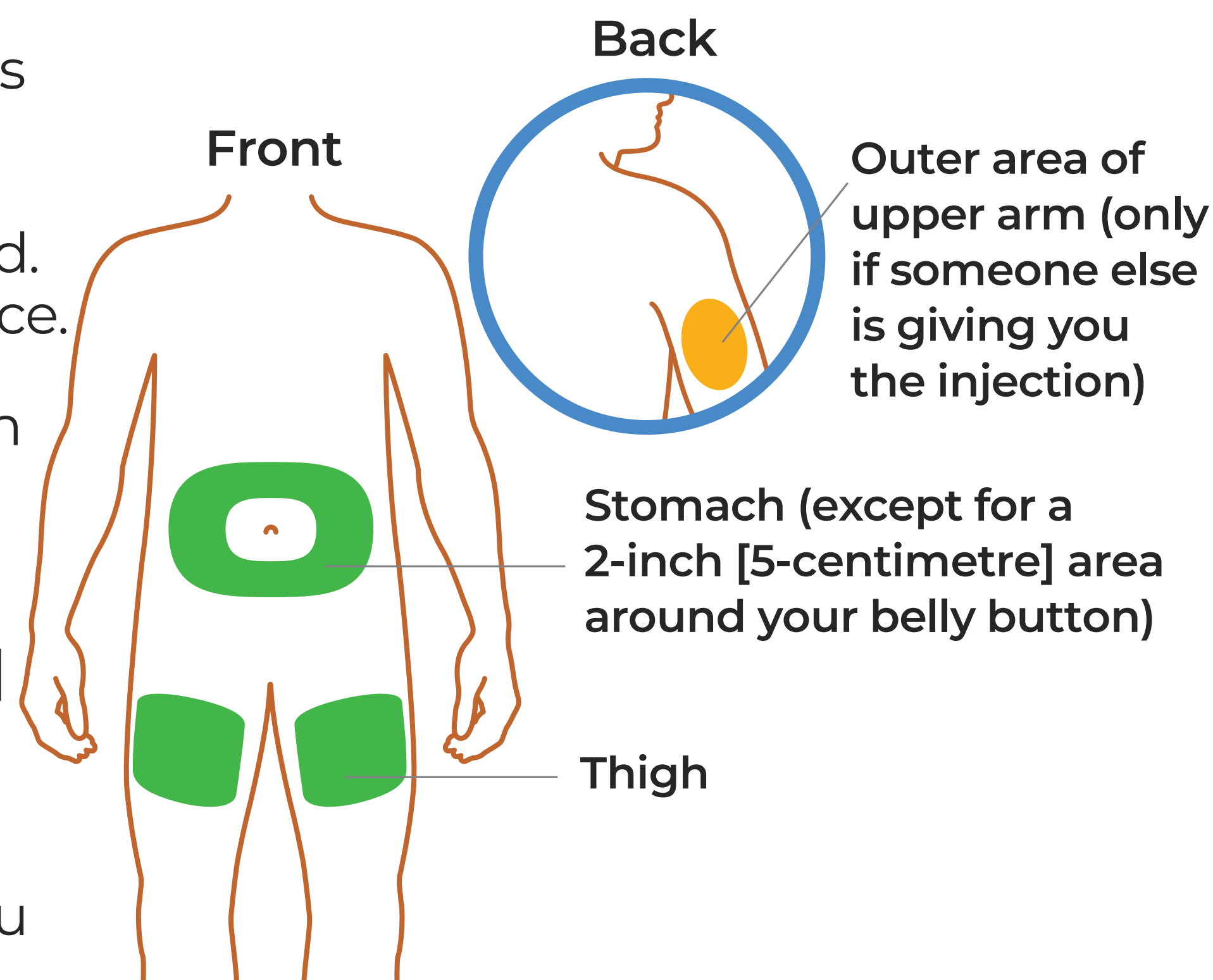

- Clean your injection site with an alcohol wipe and let your skin dry on its own.

⚠ DO NOT touch this area again before injecting.

⚠ DO NOT inject into areas where the skin is tender, bruised, red, or hard.

- Avoid injecting directly into raised, thick, red, or scaly skin patch or lesion, or areas with scars or stretch marks.

Tips!

- Pinch or stretch the skin to create a firm surface area about 2 inches (5 centimetres) wide.
- Consider an injection site that has a thicker fatty layer.
- Some people keep a diary or notes in their mobile phone to track their injections.


NEED HELP?  You can call the RepathaREADY® Patient Support Program for assistance at **1-888-Repatha (1-888-737-2842)**, Monday to Friday, 8:00 am to 8:00 pm EST.

 **Visit Repatha.ca**

GET READY | REMOVE THE CAP FROM THE AUTOINJECTOR

Pull the orange cap straight off only when you are ready to inject

If you have questions, seek help by referencing the **Patient Medication Information** leaflet or calling the RepathaREADY® Patient Support Program for assistance at **1-888-Repatha (1-888-737-2842)**, Monday to Friday, 8:00 am to 8:00 pm EST. **Or visit Repatha.ca**

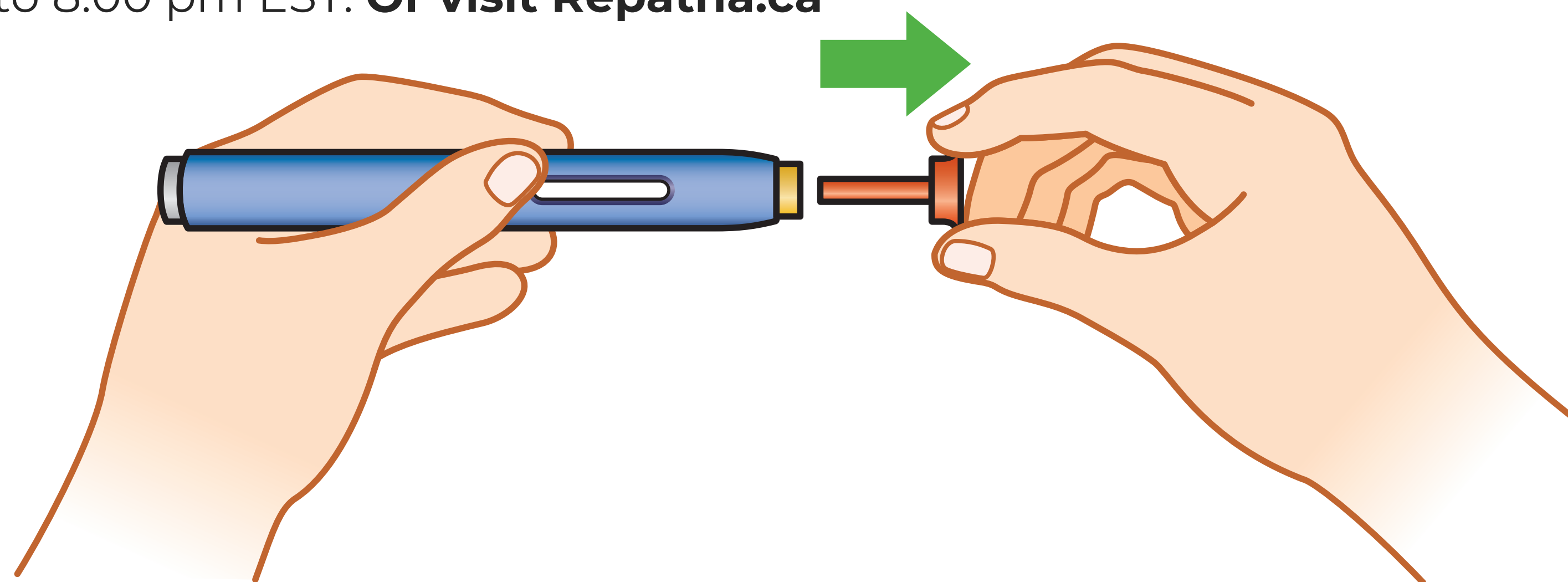


Use the autoinjector
within 5 minutes after
removing the orange cap

- ⚠ **DO NOT** twist, bend, or wiggle the orange cap.
- ⚠ **DO NOT** take the orange cap off until you are ready to inject, in order to prevent the medicine from drying out.
- ⚠ **DO NOT** put the orange cap back onto the autoinjector once it has been removed.
- ⚠ **DO NOT** put fingers into the yellow safety guard.

Please see important safety information on page 23. Refer also to the full Patient Medication Information leaflet, which is provided with your medication.


GET READY | PINCH OR STRETCH THE SKIN TO CREATE AN AREA ABOUT 2 INCHES (5 CENTIMETRES) WIDE

STRETCH METHOD 2 inches (5 cm)

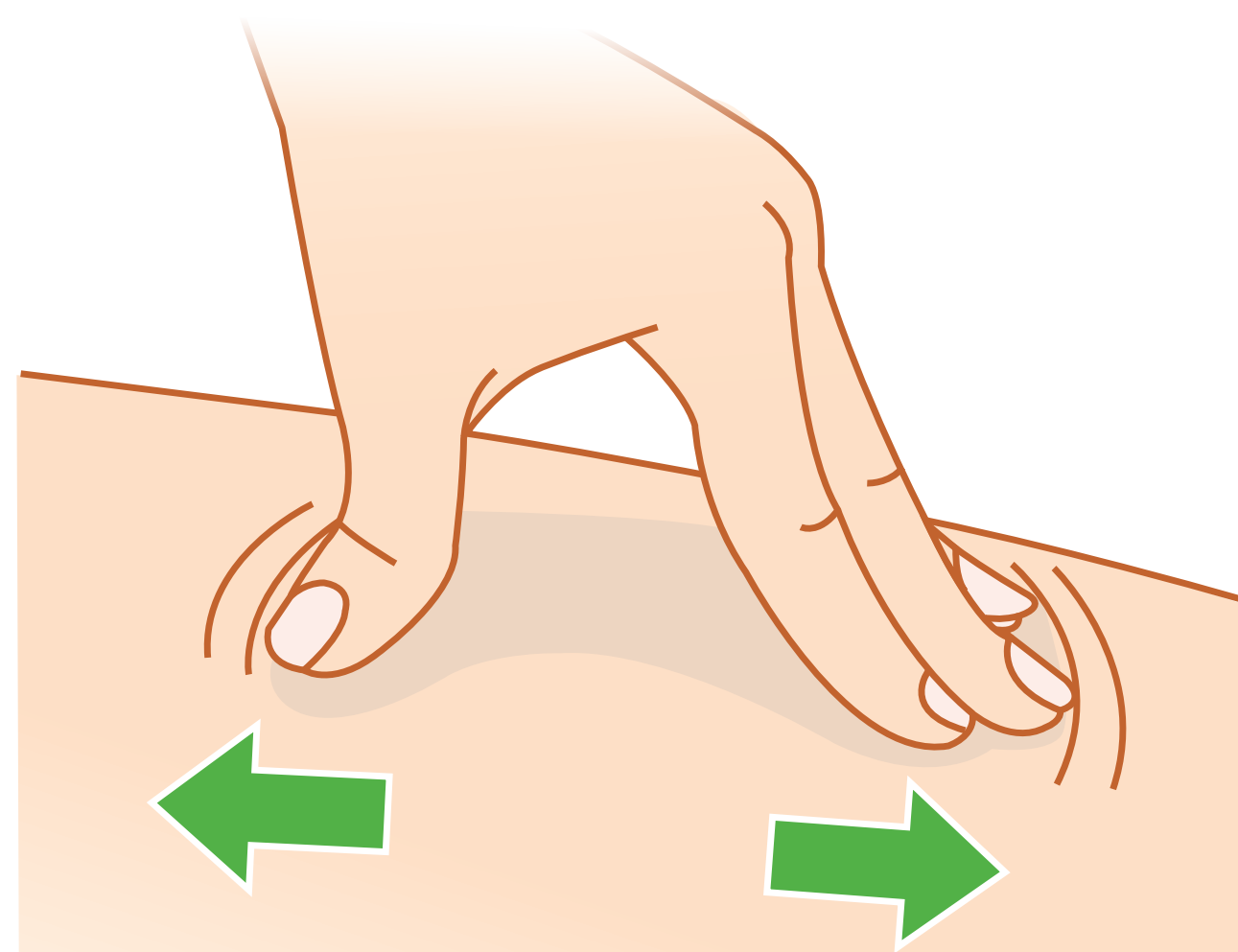

Stretch the skin firmly by moving your thumb and fingers in opposite directions, creating an area about **2 inches (5 centimetres) wide**.

PINCH METHOD 2 inches (5 cm)

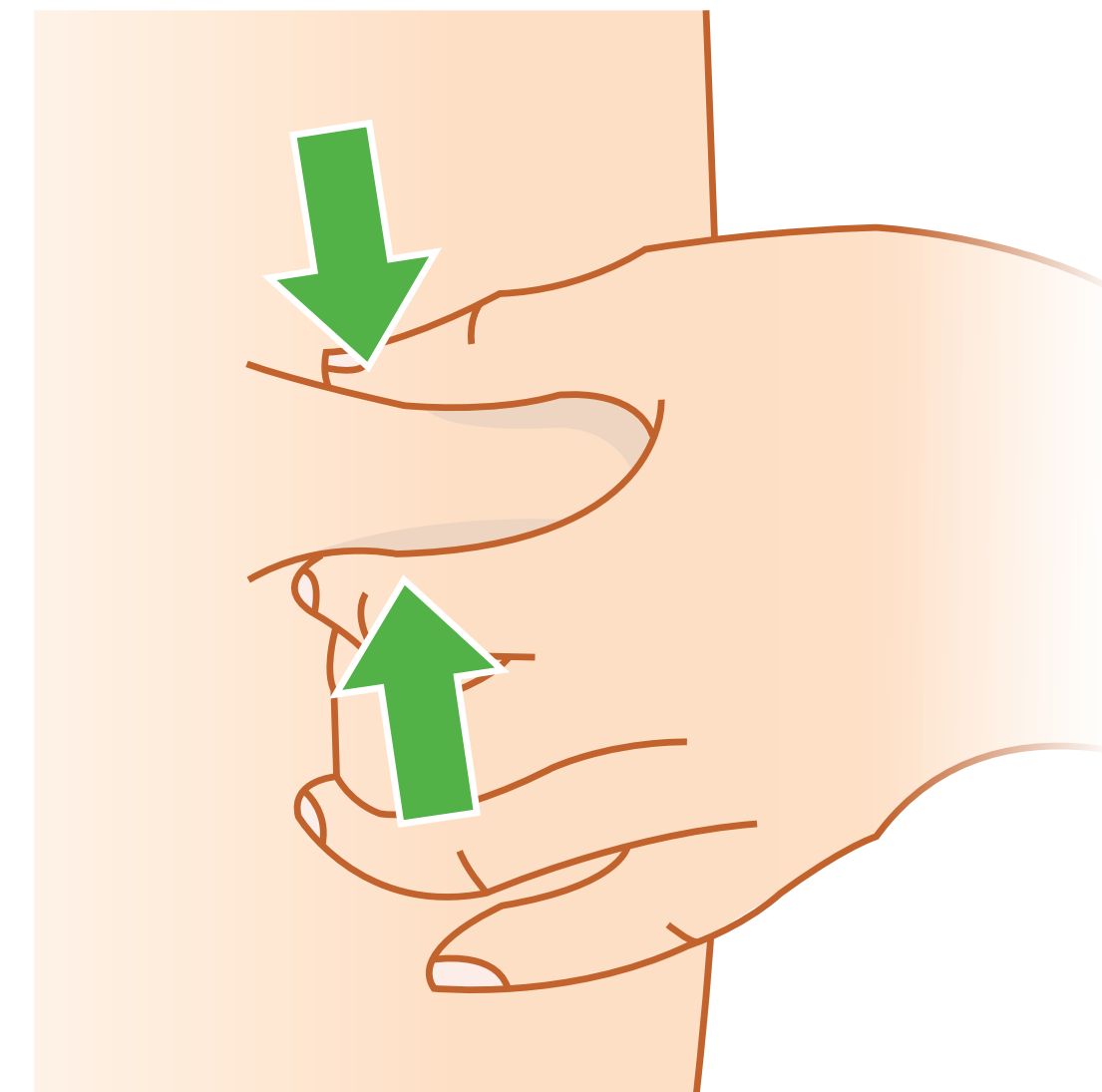


Pinch the skin firmly between your thumb and fingers, creating an area about **2 inches (5 centimetres) wide**.

 It is important to keep skin stretched or pinched throughout the injection.


NEED HELP?  You can call the RepathaREADY® Patient Support Program for assistance at **1-888-Repatha (1-888-737-2842)**, Monday to Friday, 8:00 am to 8:00 pm EST.

 **Visit Repatha.ca**


INJECT | PLACE THE SAFETY GUARD ON THE SKIN AT A 90-DEGREE ANGLE

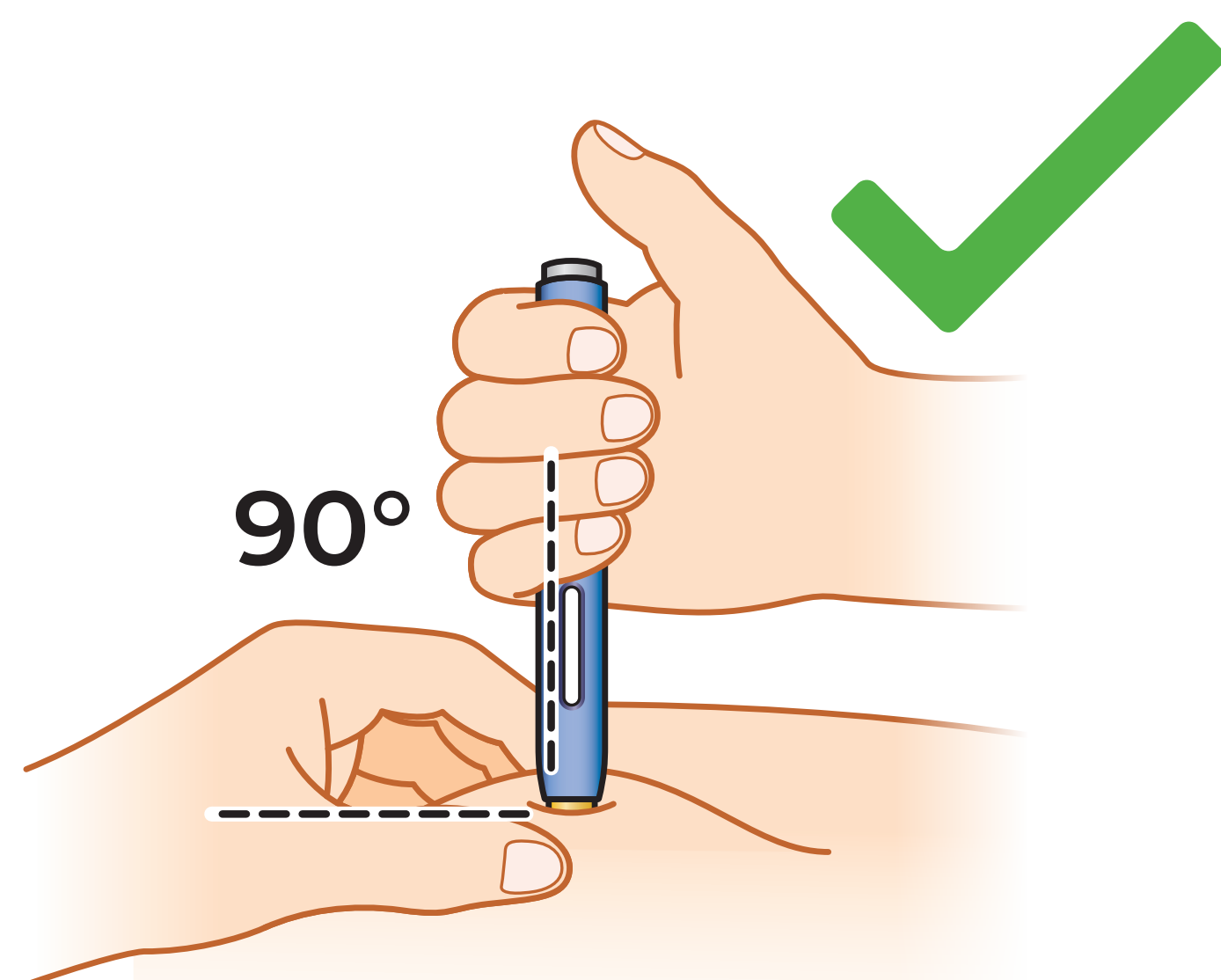

PLACE autoinjector **straight down** at 90 degrees onto the skin.

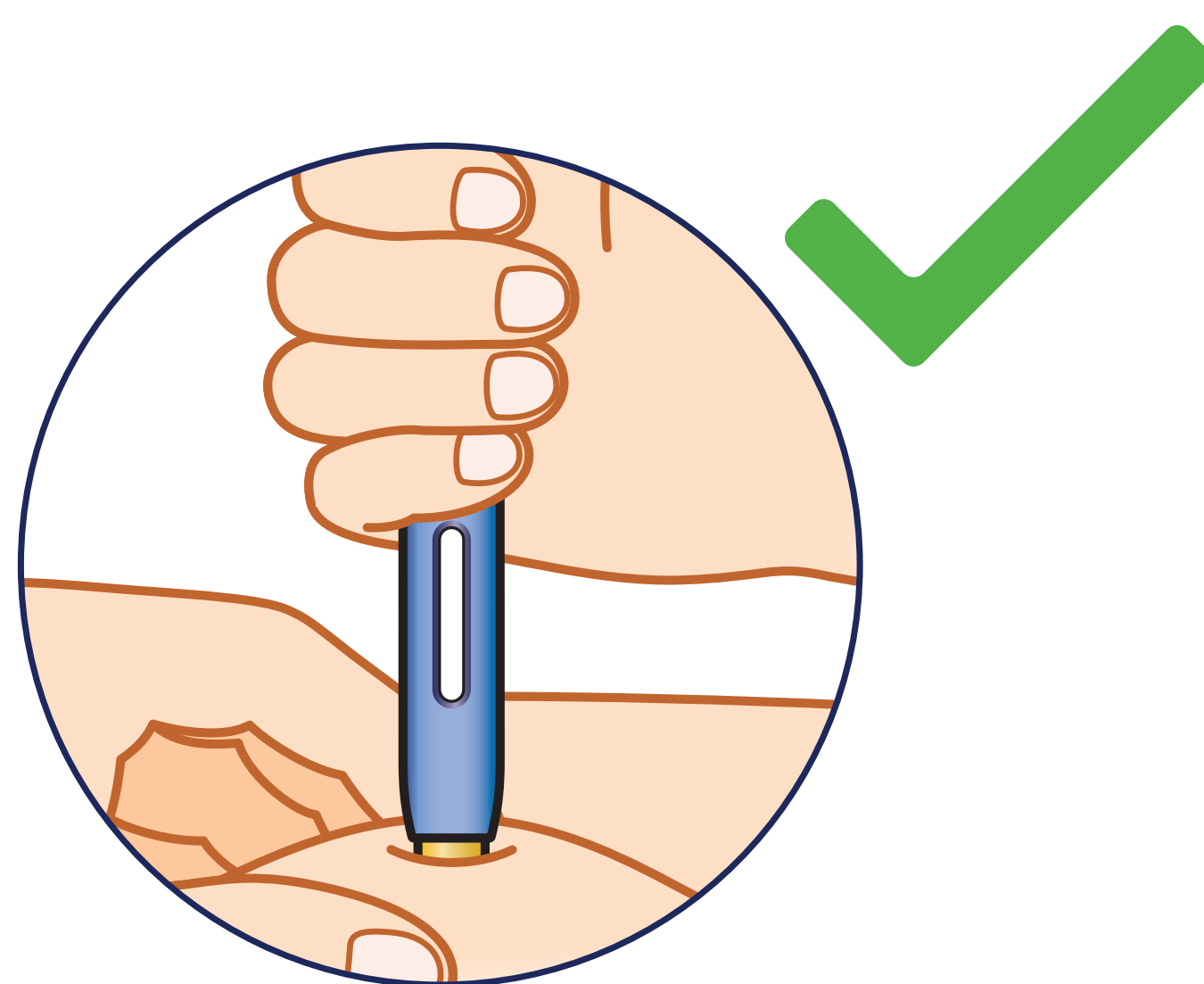

Be sure you can see the entire medicine window and that your fingers don't cover the window.

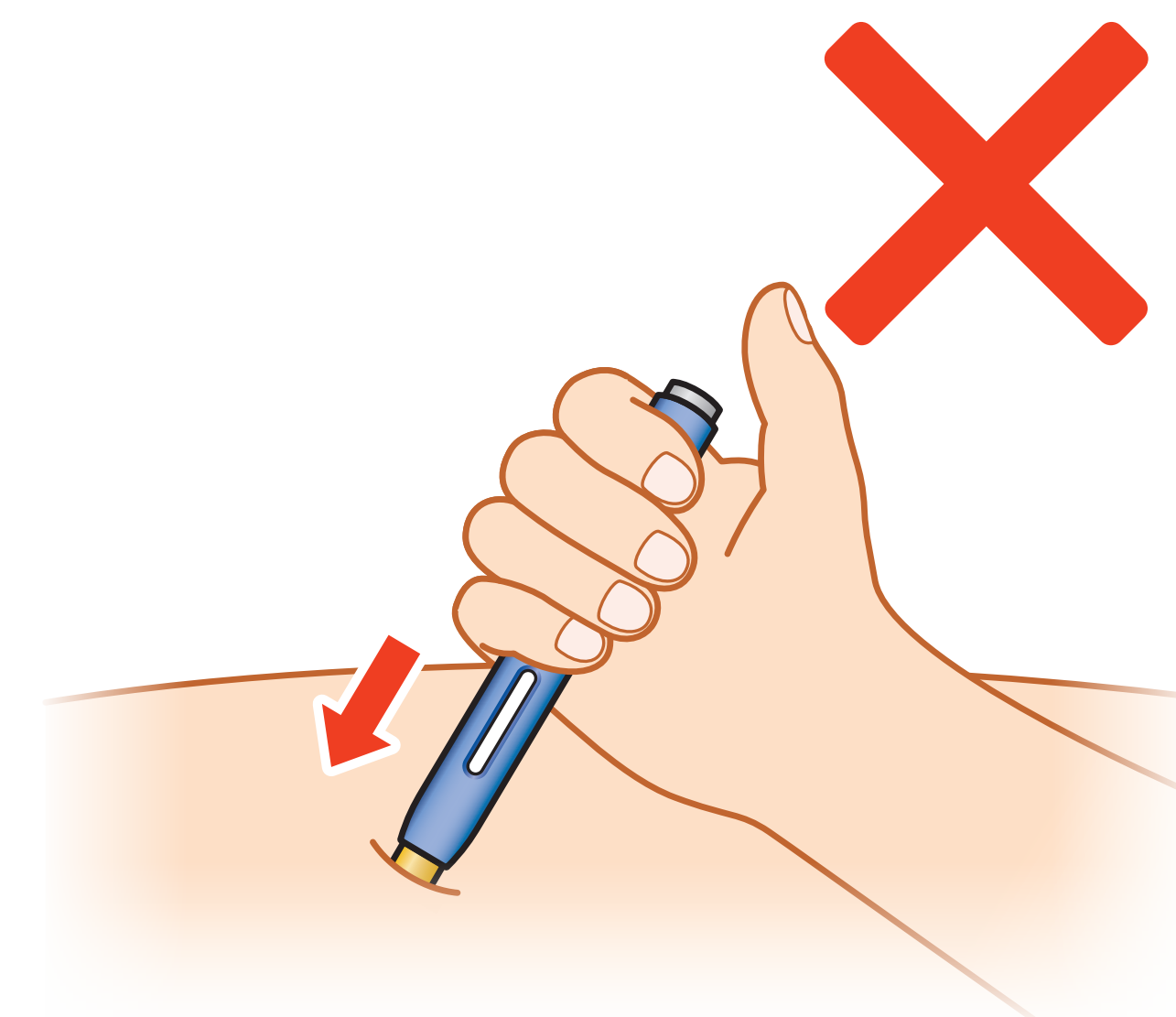


⚠ DO NOT apply the autoinjector to the skin at an angle. You may not be able to make full contact with the skin, and that can prevent the autoinjector from working properly.

- ⚠ DO NOT** touch the grey start button yet.
- ⚠ DO NOT** change positions after the injection has begun.

Please see important safety information on page 23. Refer also to the full Patient Medication Information leaflet, which is provided with your medication.

INJECT | PUSH AUTOINJECTOR FIRMLY ONTO SKIN AND HOLD

1. To start the injection, **PUSH** the autoinjector firmly onto the skin until the yellow safety guard is no longer visible

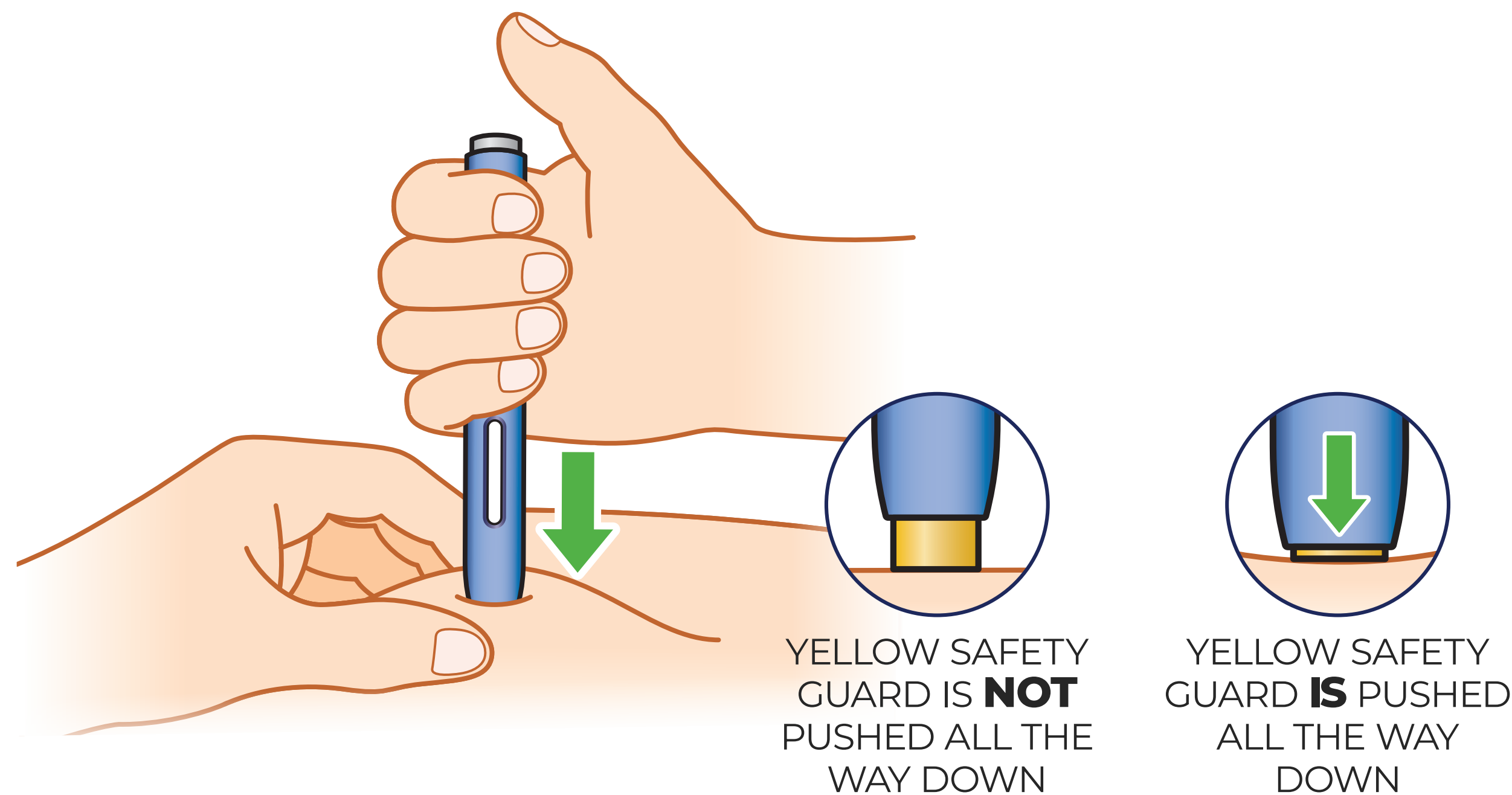

PUSH straight down and **HOLD**

- Push the autoinjector onto skin and depress the guard.
- Make sure the yellow safety guard is fully pushed down before the next step.
- **HOLD** until injection completes.


- ⚠ The **GREY START BUTTON** will work only after the yellow safety guard has been pushed all the way down.
- ⚠ Keep pushing straight down throughout the entire injection.

NEED HELP?  You can call the RepathaREADY® Patient Support Program for assistance at **1-888-Repatha (1-888-737-2842)**, Monday to Friday, 8:00 am to 8:00 pm EST.

 **Visit Repatha.ca**

INJECT | PRESS GREY START BUTTON TO BEGIN THE INJECTION

2. Next, press the grey start button to start the flow of the medicine

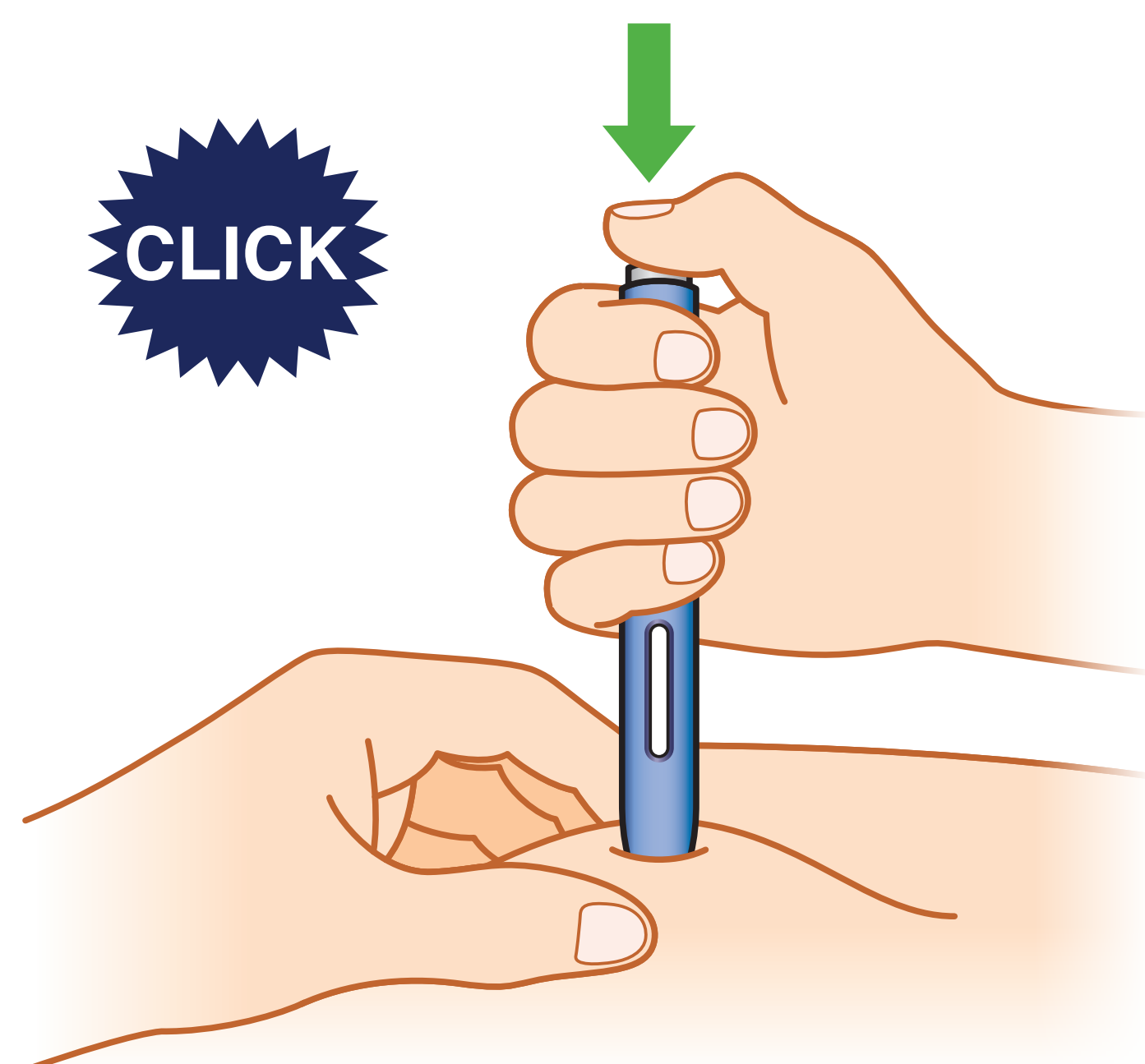

PRESS grey start button until you hear a click.

- **KEEP HOLDING** the autoinjector down with the same pressure throughout the injection.
- **GREY START BUTTON** will work only after the yellow safety guard has been fully pushed down.


⚠ If you do not hear a click after pressing the grey start button, and the window is still fully clear, ensure that the safety guard is pushed all the way down first. Then try to press the grey start button again.

⚠ These steps need to be completed in this order for a successful injection.

Please see important safety information on page 23. Refer also to the full Patient Medication Information leaflet, which is provided with your medication.

INJECT | WATCH AND CONFIRM MEDICINE WINDOW TURNS **YELLOW**

3. Finally, watch and confirm the medicine window turns **yellow** to indicate the successful completion of the injection

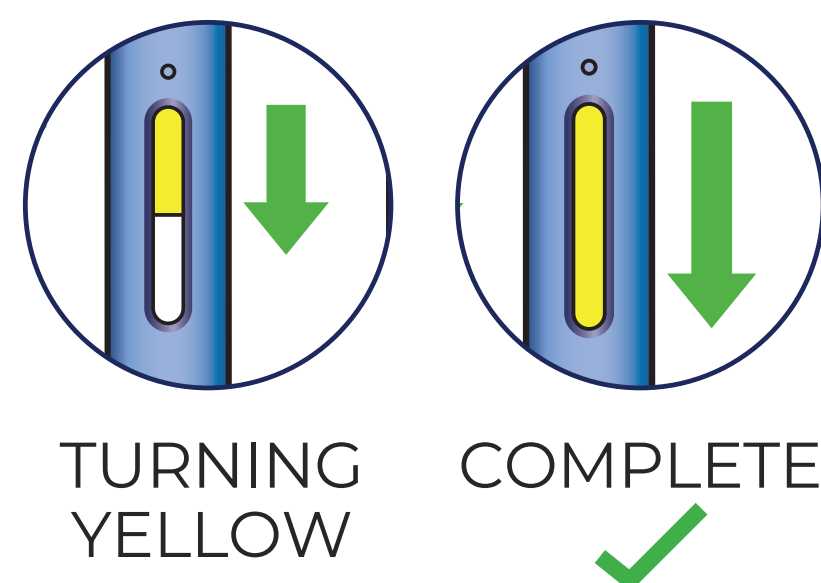

WATCH AND CONFIRM

window is fully **yellow**

- Continue holding the autoinjector down on your skin until the medicine window turns fully **yellow** to ensure all the medicine has been injected.
- It is okay to lift your thumb.
- The injection can take up to **15 seconds** to complete.
- Fully **yellow** means it's DONE!
- Window turns from clear to **yellow** when the injection is done. You may hear a **second click**. (Note: after you remove the autoinjector from your skin, the needle will be automatically covered.)
- If you did not hear a **second click** after pushing the autoinjector down on your skin for about 15 seconds, the **yellow** window still confirms the injection is done.

★ CONGRATULATIONS ON REVIEWING THE INJECTION PROCESS!

Continue to practice the steps **PUSH, PRESS, WATCH AND CONFIRM** to help build confidence. Start to make these steps a part of your Repatha® treatment routine, and be sure to contact your care team if you have any questions or concerns.

NEED HELP?




You can call the RepathaREADY® Patient Support Program for assistance at **1-888-Repatha (1-888-737-2842)**, Monday to Friday, 8:00 am to 8:00 pm EST.



Visit [Repatha.ca](https://www.Repatha.ca)

STORAGE AND DISPOSAL

Refrigeration guidelines:

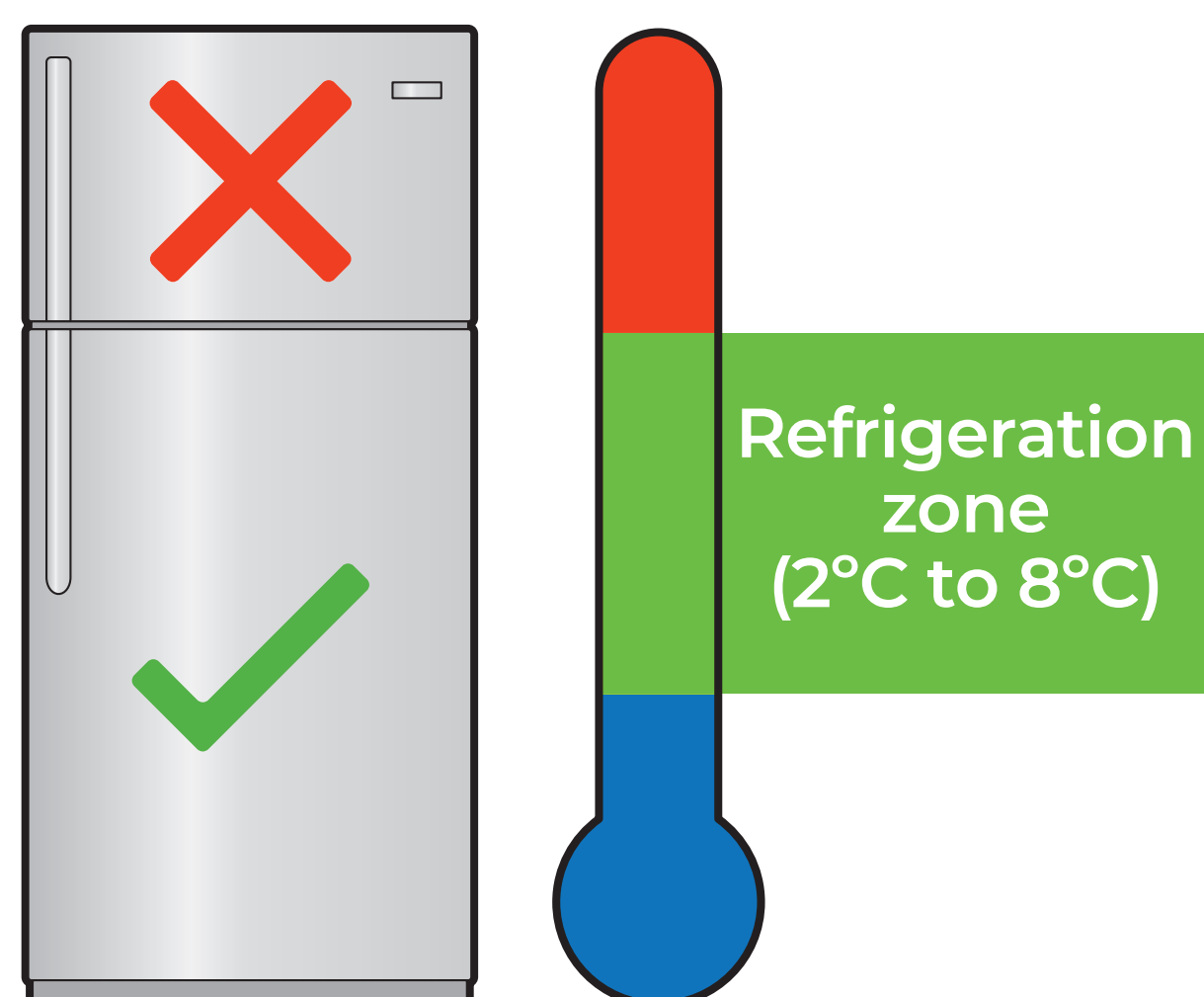

⚠ **DO NOT** freeze autoinjector.

- When you receive the autoinjector, be sure to place it directly into the refrigerator.
- Store the autoinjector in the refrigerator (2° to 8°C) until ready to use.
- Store the autoinjector in original carton to protect from light or physical damage.
- Only remove the autoinjector you plan to use.

Disposal guidelines:

- Put the used autoinjector and orange cap in a sharps disposal container right away after use.

⚠ **DO NOT** throw away the autoinjector or orange cap in your household trash.

⚠ **DO NOT** put the orange cap back on.

⚠ **DO NOT** touch the yellow safety guard.




Please see important safety information on page 23. Refer also to the full Patient Medication Information leaflet, which is provided with your medication.

CONGRATULATIONS

on reviewing the injection process!

Continue to practice the steps **PUSH, PRESS, WATCH AND CONFIRM** to help build confidence. Start to make these steps a part of your Repatha® treatment routine, and be sure to contact your care team if you have any questions or concerns.



NEED HELP?  You can call the RepathaREADY® Patient Support Program for assistance at **1-888-Repatha (1-888-737-2842)**, Monday to Friday, 8:00 am to 8:00 pm EST.

 **Visit Repatha.ca**

RESOURCES

In this section, we will discuss a variety of helpful resources available to you that are just a click or phone call away.



Please see important safety information on page 23. Refer also to the full Patient Medication Information leaflet, which is provided with your medication.

GETTING
STARTED

INJECT

RESOURCES

FAQ

TRAVEL
GUIDANCE

IMPORTANT
SAFETY INFORMATION

SURECLICK AUTOINJECTOR PATIENT SUPPORT IS AVAILABLE

Live support



RepathaREADY® Patient Support Program is ready to help.

**Call 1-888-Repatha
(1-888-737-2842), Monday to Friday,
8:00 am to 8:00 pm EST.**

Repatha® SureClick autoinjector demonstration video



Hold your smartphone camera over the QR code.

The Repatha® SureClick autoinjector demonstration video can also be viewed at
Repatha.ca

Additional resources



Visit Repatha.ca

To enter, you will need the SureClick autoinjector
DIN: 02446057.

NEED HELP? 

You can call the RepathaREADY® Patient Support Program for assistance at
1-888-Repatha (1-888-737-2842), Monday to Friday, 8:00 am to 8:00 pm EST.



Visit Repatha.ca

FAQ

TEMPERATURE CONTROL AND STORAGE:

What is the best way to store my SureClick autoinjector? Store in the refrigerator (2°C to 8°C) in its original carton.

⚠ DO NOT freeze.

I left my autoinjector outside of the refrigerator. Is this okay? Your autoinjector can be stored at room temperature (up to 25°C) for up to **30 days** in its original carton. Do not return to refrigerator once it is removed from the refrigerator and reaches room temperature.

Room temperature:
Up to 25°C Up to 30 days




ROOM TEMPERATURE GUIDELINES:

Do I really have to wait 30 minutes for the autoinjector to reach room temperature before injecting?

Yes. This is important for administering the entire dose and helps minimize discomfort. Repatha® may take longer to inject if it has not reached room temperature. Do not heat the autoinjector. Let it come to room temperature naturally.

Why do I have to use the autoinjector within 5 minutes after I remove the cap?

If you wait longer than 5 minutes, the medicine can dry out and clog the needle, resulting in failure to start the injection.

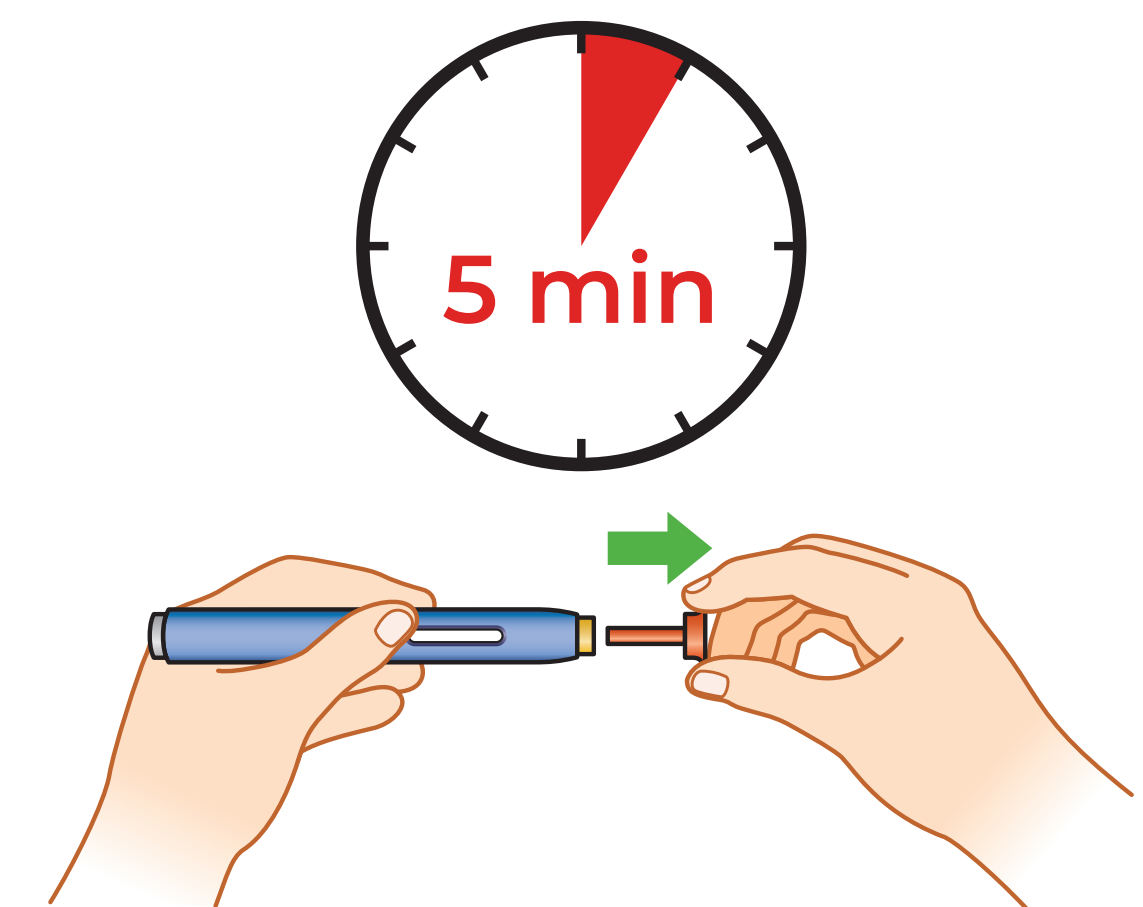

Please see important safety information on page 23. Refer also to the full Patient Medication Information leaflet, which is provided with your medication.

GETTING
STARTED

INJECT

RESOURCES

FAQ

TRAVEL
GUIDANCE

IMPORTANT
SAFETY INFORMATION

FAQ

INJECTION SITE QUESTIONS:

Why do I need to stretch or pinch the skin? You need to **STRETCH** or **PINCH** the skin to create an area 2 inches (5 centimetres) wide **to maintain a firm base** throughout the entire injection.

Can I move the autoinjector around on my skin? Yes, it's okay to move **BEFORE** you **PUSH** the grey start button.


⚠ DO NOT relocate the autoinjector **AFTER** you **PUSH** the grey **START** button, as you may not receive the full dose of medicine.

START BUTTON QUESTIONS:

Do I press and release the grey start button or do I press and hold down the grey start button during the injection? If you need to lift your thumb, it's okay. Avoid lifting the autoinjector until the injection is complete.

What will happen if I press the grey start button before I am ready to do the injection? The autoinjector will not work and will not dispense medicine, unless the yellow safety guard is pushed all the way down on the skin prior to the grey start button activation.

What if I am unable to push down the grey start button? Check to see that the yellow safety guard is no longer visible. If the yellow safety guard is not depressed fully, the grey start button may not work. You will hear a **CLICK** when the grey start button has been activated correctly.

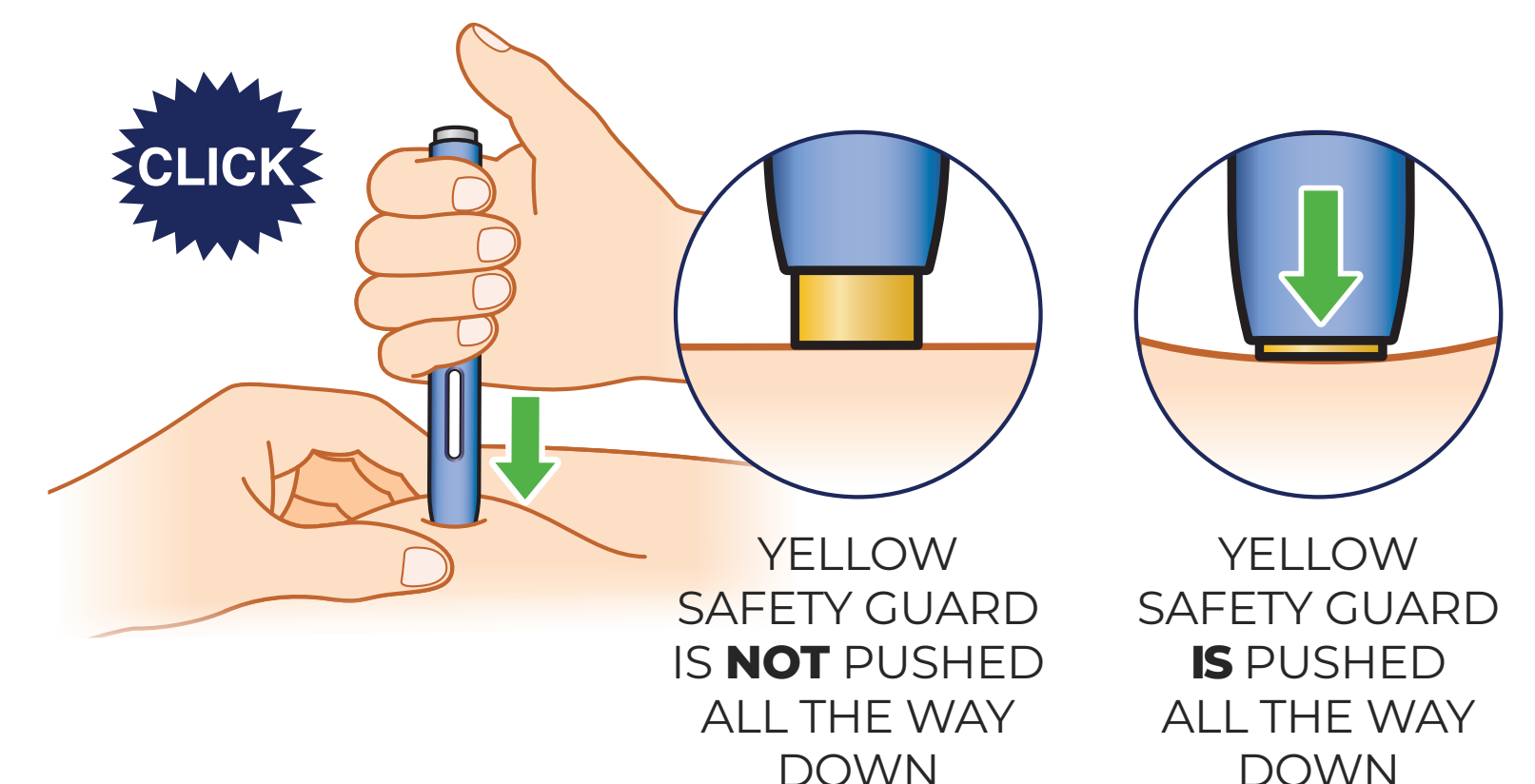

NEED HELP?  You can call the RepathaREADY® Patient Support Program for assistance at **1-888-Repatha (1-888-737-2842)**, Monday to Friday, 8:00 am to 8:00 pm EST.

 **Visit Repatha.ca**

TRAVEL GUIDANCE

GENERAL TRAVEL GUIDANCE

Yes, you can travel. You don't have to miss any of your treatments. In fact, it's important that you don't miss your injection when travelling.

- Always keep your medicine with you to ensure it is kept within the safe temperature zone and is not broken.

⚠ **DO NOT** store the autoinjector in extreme heat or cold. For example, avoid storing the autoinjector in your vehicle's glove box or trunk.

⚠ **DO NOT** shake the autoinjector.

⚠ **DO NOT** use the autoinjector if it has been dropped on a hard surface. Part of the autoinjector may be broken even if you cannot see the break. Use a new autoinjector, and call **1-888-Repatha (1-888-737-2842)**.

Please see important safety information on page 23. Refer also to the full Patient Medication Information leaflet, which is provided with your medication.

SHORT TRIPS

- The autoinjector can be kept at room temperature (up to 25°C) in the original carton for up to 30 days without refrigeration.

⚠ Remember that once the autoinjector reaches room temperature, you should not put it back in the refrigerator.

Important:
Keep the autoinjector in the original carton.



LONGER TRIPS

- Reach out to your pharmacist and/or doctor to discuss other options such as filling your prescription at your desired destination.



GETTING
STARTED

INJECT

RESOURCES

FAQ

TRAVEL
GUIDANCE

IMPORTANT
SAFETY INFORMATION

TRAVEL GUIDANCE


CAR AND AIR TRAVEL GUIDANCE

Travelling by car:

⚠ **DO NOT** leave the autoinjector in your car if exposed to extreme heat, extreme cold, or direct sunlight.

⚠ **DO NOT** store the autoinjector in your vehicle's glove box or trunk.


- Once the autoinjector reaches room temperature, do not put it back in the refrigerator.

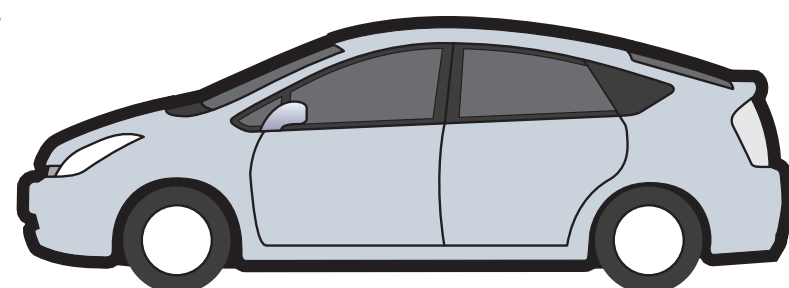

Travelling by airplane:

Before you travel:

- Contact your airline about the carry-on policy for needles, syringes, or injection devices. Security regulations may change.
- Research the injection device requirements of other countries you may visit.
- Bring a copy of your prescription or a doctor's note to show airport security.

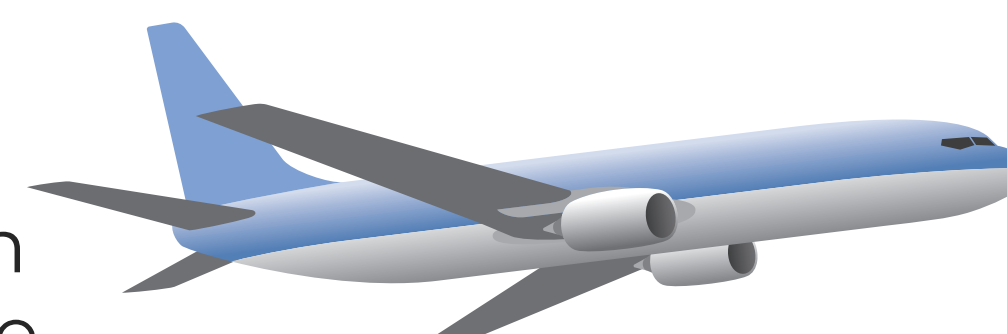

Airport security considerations:

- Notify security screeners that you are carrying a sharps container.
- Consider requesting a “hand check” of your medicine kit to avoid any delays.
- Be prepared to display, handle, and repack your own medicine and supplies in order to prevent contamination or damage.



For information on airline travel with Repatha®, please refer to: Canadian Air Transport Security Authority (CATSA) www.catsa-acsta.gc.ca/

- If you can carry-on Repatha®, ensure the **prescription (Rx) label** is attached.
- If you need extra help or have concerns about airport security, please call **1-888-294-2202**.
- For flights originating **outside of Canada**, check with your air carrier or travel agent for any restrictions.

NEED HELP? 

You can call the RepathaREADY® Patient Support Program for assistance at **1-888-Repatha (1-888-737-2842)**, Monday to Friday, 8:00 am to 8:00 pm EST.



Visit Repatha.ca

IMPORTANT INFORMATION ABOUT REPATHA®

Do not use Repatha® if you have ever had an allergic reaction to Repatha® or any of the ingredients in Repatha®.

Talk to your healthcare professional if you have allergies, or are allergic or sensitive to rubber or latex, as the needle cap on the SureClick autoinjector contains dry natural rubber.

To help avoid side effects and ensure proper use, talk to your healthcare professional before you take Repatha®. Talk about any health conditions or problems you may have.

If you use Repatha® together with a statin and other cholesterol-lowering medicines, please read the package leaflet of that medicine.

Children and adolescents

The use of Repatha® has been studied in children 10 years of age and older being

treated for heterozygous or homozygous familial hypercholesterolemia.

The use of Repatha® has not been studied in children under 10 years of age or in children with other types of hypercholesterolemia.

Other medicines and Repatha®

Tell your doctor or pharmacist about all the medications you are taking or have recently taken, including medicines obtained without a prescription such as drugs, vitamins, minerals, natural supplements or alternative medicines.

Pregnancy and breastfeeding

Repatha® has not been tested in pregnant women. It is not known if Repatha® will harm your unborn baby. If you are trying to get pregnant or become pregnant while taking Repatha®:

- Inform your doctor.

- If you are also taking a statin along with Repatha®, stop taking Repatha® and read the package leaflet of the statin that you are taking with Repatha®.

It is not known whether Repatha® is found in breast milk. It is important to tell your doctor if you are breastfeeding or plan to do so. Your doctor will then help you decide whether to stop breastfeeding, or whether to stop taking Repatha®, considering the benefit of breastfeeding to the baby and the benefit of Repatha® to the mother.

Please see important safety information on page 23. Refer also to the full Patient Medication Information leaflet, which is provided with your medication.

GETTING
STARTED

INJECT

RESOURCES

FAQ

TRAVEL
GUIDANCE

IMPORTANT
SAFETY INFORMATION

WHAT IS REPATHA® AND WHAT IS IT USED FOR?

Repatha® is a medicine used to lower levels of cholesterol.

Why have I been prescribed Repatha®?

Repatha® is used:

- along with diet and in combination with usual therapy, including cholesterol-lowering medications called statins, to reduce the risk of heart attack, stroke and certain heart procedures (to restore blood flow to the heart) in adult patients who have cardiovascular disease by further lowering low-density lipoprotein (LDL) cholesterol.
- along with diet, alone or together with other cholesterol-lowering

therapies, in adults with primary hyperlipidemia (high LDL cholesterol level in your blood) to reduce LDL cholesterol.

- along with diet and other cholesterol-lowering therapies, to reduce LDL cholesterol in people 10 years and older who need additional lowering of the high LDL cholesterol in their blood because of a condition that runs in their family (heterozygous familial hypercholesterolemia [HeFH]).
- along with diet and other cholesterol-lowering therapies, to reduce LDL cholesterol in people 10 years and older who need

additional lowering of the high LDL cholesterol in their blood because of a condition that runs in their family (homozygous familial hypercholesterolemia [HoFH]).

NEED HELP?  You can call the RepathaREADY® Patient Support Program for assistance at **1-888-Repatha (1-888-737-2842)**, Monday to Friday, 8:00 am to 8:00 pm EST.

 **Visit Repatha.ca**



**RepathaREADY® Patient Support Program
is ready to help.**

**Call 1-888-Repatha (1-888-737-2842),
Monday to Friday, 8:00 am to 8:00 pm EST.**

Visit Repatha.ca
(SureClick autoinjector
DIN: 02446057) or scan
the QR code.



© 2024 Amgen Canada Inc. All rights reserved.
Repatha®, RepathaREADY® and SureClick® are registered
trademarks of Amgen Inc., used with permission.
AMGEN ENTRUST® is a registered trademark of Amgen Inc.

AMGEN Entrust®

RepathaREADY®
PATIENT SUPPORT PROGRAM



AMGEN
Cardiovascular

CAN-145-1123-80008-24E